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**Q010004NO003\_V03 2026/05/11 – INSTRUCTIONS FOR USE RANGE SOKLEAN – English**

**WARNING**

In order to obtain a result in compliance with the possibilities of this accessory, it is mandatory to respect those instructions.

**INDICATIONS**

The SOKLEAN is an accessory for dental medical devices specifically developed to maintain those medical devices sterile and to present them into the sterile operative field. SOKLEAN configurations allow the medical device prehension by hand or with an instrument, without any contamination risk.

**PRODUCT DESCRIPTION**

The SOKLEAN is a holder system for an aseptic recovery designed to maintain dental implants in different type packaging solutions like pouches, blisters and SoBliss.

The SOKLEAN is composed of 4 parts. Three in injected plastic named the body, the lid and the slider, and a last one machined in steel named the collar. The body is the main part which receive others one with different interactions. The implant screw is maintained in the holder by the collar, press fitted in the top of the body, this inner diameter is adapted to the outer implant diameter to prevent many directions movements. By gravity, the implant screw, rests on his own healing screw or a toe. To block the axis movement, the rotated lid has 3 different positions. One allows this the body to close the holder and protect the implant and both others open (forward / backward) to pick up the implant screw.

The final recovery of the implant screw is picked up with the adapted customer tool or drop on the sterile field if necessary. The healing screw is clipped on the slider, which can manually move along the body (in only one direction i.e. to the top) to approach closer to the collar for an easier recovery.

**CHARACTERISTICS**

- Biocompatible materials (without unauthorized leachable or non-leachable products)
- The accessory is compatible with implantable or non-implantable medical devices.

**SYMBOLS USED ON THE PRODUCT**

Reference

Batch number

Single use. Do not Reuse

Do not use if damaged

Manufacturer address

Manufacturing date

Non-sterile


**BASIC PRECAUTIONS FOR PRODUCT PRESERVATION**

The product has been validated for a usage, which includes gamma irradiation between 25 and 45 kGy (sterilization mode of the client) without losing its properties. Any other usage, that differs from the intended one defined by SELENIUM MEDICAL during its validation, is under the final user responsibility.


**INSTRUCTIONS FOR USE**

According to the chosen configurations, assembly modes of SOKLEAN components can differ. The picture below shows an example :


**Medical device mounting in the SOKLEAN :**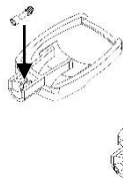


1- Insert the healing screw in the slider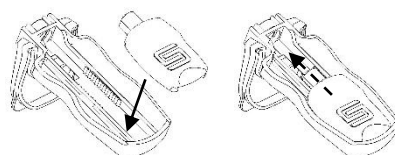

2 – Put the slider in the body and adjust the position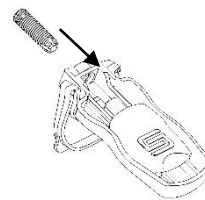


3 – Insert the implant screw

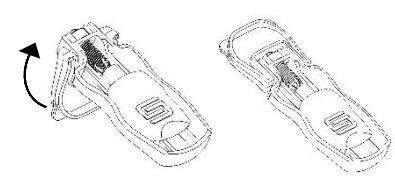
4 – Close the lid


**Medical device recovering from the SOKLEAN:**

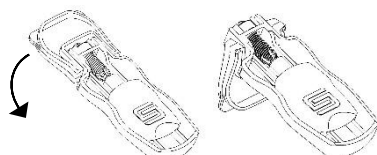1 – Open the lid

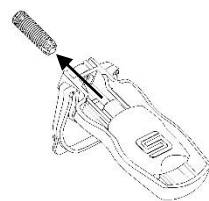2 – Pick up the implant screw with adapted tool

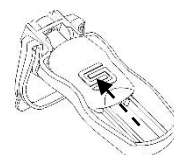3 – Ride up the slider

4 – Unscrew the healing screw

**PRECAUTIONS OF USE**

- The product shall not be modified. It can only be used if its integrity is preserved.
- It is necessary to ensure the good consistency of the medical device in the holding system before handling.
- Do not disassemble the different parts making up the SOKLEAN, risk of falling the implant.
- Activate the slide only once the implantable screw has been recovered.

**SECONDARY EFFECTS**

No secondary effect has been observed yet.

**COUNTER-INDICATIONS**

- Any other use of this accessory than the ones mentioned in the INDICATIONS section is a counter-indication.
- Non-implantable device: DO NOT IMPLANT – DO NOT REUSE.

**SPECIFIC STORAGE CONDITIONS**

SOKLEAN products must be stored (as separated components, assembled components or into the operative room) away from excessive heat and direct light. Ambient temperature is preconized. SOEASY products must be protected from high humidity and stored into temperate, dry and clean conditions.

SELENIUM MEDICAL declines all responsibility if above storage conditions are not respected.

**WASTE DISPOSAL**

The elimination of medical waste must comply with the applicable legislation. The professional who produces waste will be responsible for sorting waste which carries a risk of infection and waste which is compatible with household waste.

**FACTORS THAT CAN COMPROMISE THE PRODUCT USE**

- Not respecting the usage process of this accessory can lead to sterility loss of the medical device that it contains.
- Not respecting the storage conditions can cause physical modifications of the accessory, which can lead to an impossibility of a simple and secure use.
- Components with the same color index must be assembled, no mixing, except for the references listed below :

		LIDL				SLIDER			
		HDA2PGN022024007	HDA2PGN022024007L	HDA2PGN022024007B	HDA2PGN022024007Y	HDA2PGN02801600X	HDA2PGN02801600XL	HDA2PGN02801600XB	HDA2PGN02801600XY
BODY	HDA2PGN050024007	X	V	X	X	X	V	X	X
	HDA2PGN050024007L	V	V	V	V	V	V	V	V
	HDA2PGN050024007B	X	V	X	X	X	V	X	X
	HDA2PGN050024007Y	X	V	X	X	X	V	X	X